NOVAFON SONIC WAVE INSTRUMENTS

User manual

TABLE OF CONTENTS

English	04
Français	14
Dansk	25
Deutsch	35
Español	45
Italiano	55
Magyar	65
Nederlands	75
Svenska	85

TABLE OF CONTENTS

1. **FOREWORD**
2. **INSTRUMENT DESCRIPTION**
 - 2.1. Instrument types
 - 2.1.1. SK2 chrome-plated - The Versatile
 - 2.1.2. SK1 gold-plated - The Powerful
 - 2.1.3. SK1 chrome-plated - The Original
 - 2.1.4. SK1 plastic - The "Sonossage"
 - 2.2. Accessories
 - 2.2.1. Sonic ball
 - 2.2.2. Sonic plate
3. **OPERATING INSTRUCTIONS**
 - 3.1. Initial operation
 - 3.2. Intensity regulation
 - 3.3. Massage attachments
4. **OPERATION**
5. **USAGE**
 - 5.1. Fields of application
 - 5.1.1. Wellness applications
 - 5.1.2. Applications to accompany therapy
 - 5.2. Application of liniments
 - 5.3. Contra-indications and side effects
 - 5.3.1. Contra-indications
 - 5.3.2. Side effects
 - 5.3.3. Combination with other products
6. **TECHNICAL SPECIFICATIONS**
 - 6.1. Symbols used
 - 6.2. Technical specifications
7. **SAFETY INSTRUCTIONS**
8. **MAINTENANCE AND REPAIR**
 - 8.1. Cleaning and disinfection
 - 8.2. Instructions in case of defects
 - 8.3. Safety test
 - 8.4. Disposal
9. **WARRANTY**
10. **CUSTOMER SERVICE**

1. FOREWORD

It is our pleasure to be able to welcome you as a NOVAFON customer. You made a good choice in purchasing the original NOVAFON sonic wave instrument. Our therapy devices are easy to use, versatile, and have proven themselves over the decades. We recommend that you read the following pages carefully in order to get maximum use from your NOVAFON sonic wave instrument and to ensure lasting enjoyment.

2. INSTRUMENT DESCRIPTION

NOVAFON sonic wave instruments are available in different versions but they all operate according to the same mechanical principle: a moving coil inside the device causes the head (plate or ball attachment) to vibrate. Vibrations produced by the NOVAFON sonic wave device are periodic mechanical oscillations within the audible frequency range. They are applied to the tissue vertically and have an effective range of up to 6 cm in depth.

The various devices primarily differ in terms of the number of settings (1 or 2), effective depth (4.2 or 6 cm), plating material (chrome-plated, gold-plated, or plastic), outer casing, and features (with or without back handle).

The first setting is the basic setting with which all of the devices are fitted. The SK2 chrome-plated model and SK1 gold-plated model both additionally feature a pummel/percussion setting (setting 2).

Setting 1 = Basic setting: 120 vibrations per second

Setting 2 = Pummel/Percussion: 60 vibrations per second

Each device also features a fully-adjustable intensity dial which enables fine tuning.

Two massage attachments are included in the delivery of all devices: sonic plate and sonic ball (See Chapter 2.2). The SK2 chrome-plated instrument includes a retractable, extendable handle that makes it easier to treat areas of the body which are harder to reach.

2.1. Instrument types

2.1.1. SK2 chrome-plated - The Versatile

The SK2 therapy instrument is equipped with two switch settings and a retractable extension. It is effective to a depth of 6 cm. The SK2 is particularly versatile and is suitable for all NOVAFON applications.

2.1.2. SK1 gold-plated - The Powerful

The gold-plated SK1 therapy instrument is equipped with two switch settings and is calibrated for power. This means that the vibrations it produces are stronger than those of the other devices. It is effective to a depth of 6 cm. This therapy instrument is ideally suited to massaging particularly taut muscles and treating trigger points.

2.1.3. SK1 chrome-plated - The Original

This is the original NOVAFON sonic wave instrument in its original form. It is fitted with a single switch setting and is effective to a depth of 6 cm.

2.1.4. SK1 plastic - The "Sonossage"

The SK1 therapy instrument is made from plastic, it comes equipped with a single switch setting and was designed for cosmetic purposes. With an effective depth of 4.2 cm it is exceptionally gentle. The SK1 plastic is available in a range of colours.

2.2. Accessories

All devices come with two interchangeable attachments: the sonic plate and sonic ball.

2.2.1. Sonic ball

The sonic ball can be used to massage parts of body that are less easily reached and to treat isolated areas. It is included in the delivery and can also be purchased separately.

2.2.2. Sonic plate

The plate attachment is used to massage body parts that are easy to pass over. It is included in the delivery and can also be purchased separately.

3. OPERATING INSTRUCTIONS

3.1. Initial operation

Insert the mains plug into a power socket which corresponds to the specified voltage. Ensure that the cable is not stretched in excess of 3 metres, as this could cause the cable to break. If necessary, use an extension cord.

The following functions are provided for the devices:

Description	SK2 chrome-plated	SK1 gold-plated	SK1 chrome-plated	SK1 plastic
Setting Standard operation	I	I		
Setting Pummel massage	II	II		
Switch off				
Intensity control				

3.2. Intensity regulation

Intensity is regulated by turning the control dial. Viewed from above, rotating the dial to

the right i.e. in a clockwise direction decreases the intensity. Correspondingly, turning the dial to the left i.e. in an anti-clockwise direction increases the intensity. We recommend you begin with a low intensity, then increase it if and when necessary. Applying pressure with the instrument does not increase its effectiveness, in fact it only impedes the vibratory performance. Optimal vibration is achieved by simply allowing the device's own weight to rest on the skin.

3.3. Massage attachments

The two attachments (sonic ball and sonic plate) can be easily interchanged by unscrewing them while applying a small amount of pressure on the device. Should accessories need to be replaced due to normal wear and tear, you can order the corresponding spare parts from your supplier.

4. OPERATION

The NOVAFON instrument produces sonic waves with mixed frequencies of between 100 and 400 Hertz (the audible frequency range) and has an effective depth of 6 cm or more. Therefore, the vibrations which are produced lie within a frequency range that can be directly felt at the point of contact and in the surrounding area. In this sense NOVAFON sonic wave therapy can be compared to vibration therapy. The NOVAFON sonic wave instrument's depth effectiveness depends on the directed force with which the oscillations or vibrations are vertically applied to the tissue and spread by means of the massage attachment. A study by the Fraunhofer Institute (IBP - report 113/84) demonstrated that the depth effectiveness of at least 6 cm (this applies to all chrome-plated and gold-plated versions of the instrument) is a measurable value. The findings of this study show that the deep penetrative effect of sonic wave treatment can be attained by both direct application to the bone and to relatively soft tissue.

5. USAGE

The NOVAFON instrument is passed over or around the area to be treated using linear strokes or in slow circular motions. Unless otherwise prescribed, treatment can be carried out twice a day (or more often if necessary) for a duration of between 15 to 20 minutes respectively. The duration of the application can be reduced when treating sensitive or very sore parts of the body. The instrument does not have to be used on bare skin. Treating areas covered by clothes or by a thin cloth allows the device to glide more smoothly. Textiles have a negligible influence on the deep penetrative effect.

5.1. Fields of application

The list of sonic wave treatment applications entails a wide range of uses for NOVAFON sonic wave instruments. They are employed by doctors, therapists and clinics as an accompaniment to therapy, but they can also be used for a range of applications in the home thanks to their ease of use. In case of serious illnesses, information on correct usage should be sought from the treating physician or therapist.

5.1.1. Wellness applications

NOVAFON sonic wave instruments are exceptionally well-suited to wellness applications such as relaxing full-body massages, invigorating foot and leg massages, and soothing face massages. For sustainable results the treatment listed here should be carried out daily; or even multiple times per day, e.g. mornings and evenings.

Useful tips for leg massages

Using the plate attachment massage from bottom to top in stroking or gently circular motions. Always move the NOVAFON in one direction. When used for leg massages the device should always be moved from bottom to top, i.e. towards the heart. This relieves strain from the legs while stimulating lymph circulation. The massage should last between 10 and 15 minutes and should include both legs.

Useful tips for back massages

Using the plate attachment move the device over the sore area either in linear strokes or in slow circular motions. During the treatment you should constantly slowly massage from top to bottom (or vice versa) to the left and right of the spine. Once you have massaged in one direction, lift the device from the skin, then recommence from where you initially began. The treatment should last between 15 and 20 minutes.

Useful tips for facial massages

Massage in circular or stroking movements. Use the sonic plate for the middle of the face. Begin massaging at the bridge of the nose: rub from here outwards over the face. Massage the whole cheek area in this way. Include the cheekbones and rub gently in the direction of the ear. Eyes: Place the NOVAFON under the eye and circle around it gently without applying direct pressure. Begin massaging the bony part around the eye sockets. Do not directly massage the eyes or the eyelids. Lips: Use the sonic ball and place the device around the edge of the lips. Slowly move the NOVAFON around the mouth in circular motions.

5.1.2. Applications to accompany therapy

Vibration therapy using NOVAFON can be effective as a supporting measure for serious illnesses, however it does not replace a visit to the doctor. Ideally, its application should be arranged with the treating doctor or therapist.

In accordance with the contra-indications (Chapter 5.3) the use of the instrument is dependent on the subjective perception of the person being treated. The device can also be used in compliance with the symptom: placed directly on or in the immediate vicinity of the aching area, on the antagonistic muscle and/or the antagonistic tendon, on trigger points near the aching area (or even acupuncture points), or contralateral to the aching area.

Applications in the context of orthopaedic indications

- Osteoarthritis (joint wear)
- Bursitis (inflammation of the bursa)

- Epicondylitis (tennis elbow)
- Fibromyalgia (fiber-muscle pain)
- Lumbar (lower) back pain
- Myalgia (muscle pain)
- Recovery of muscle performance following cruciate ligament surgery
- Pain reduction in acute and chronic musculoskeletal pain
- Tendonitis (tendon inflammation)
- Torticollis (e.g. with cervical spine syndrome)
- Cervical spondylosis (nerve root compression in the cervical spine region)

Application in the context of occupational therapy treatment plans and rehabilitation measures

- Hand therapy
- Hemiparesis (spasticity, following a stroke)
- Scar treatment
- Rehabilitation after neurological diseases
- Polyneuropathy
- Assistance in physiotherapy of stroke patients or patients with limb spasticity
- Improvement of body schema
- Improvement and preservation of mobility
- Spinocerebellar ataxia (defects in the ordered sequence of muscle movements)
- Cerebral palsy: To aid in the treatment of muscular hypertonia

Application in the context of logopaedic treatment plans

- Dysphagia
- Functional dysphonia
- Vocal cord paralysis
- Reduction of orofacial pains of a variety of causes
- Improving articulation and food intake

Further fields of applications

- Muscle regeneration after exercise
- Phantom pain
- Pain reduction for outer dermatological and cosmetic procedures
- Sinus pain
- Improvement of muscle performance and posture

5.2. Application of liniments

As a rule only apply cremes, ointments, and cosmetic/medicinal liniments to the skin after NOVAFON treatment. Using the device prior to application of these compounds stimulates blood flow, which in turns leads to a more efficient uptake of the active ingredients contained in them. If you nevertheless wish to apply ointments or cremes before NOVAFON treatment, we recommend you place a fine cloth between the instrument and skin. Otherwise the residue left by these compounds may damage the device.

5.3. Contra-indications and side effects

In case of doubt consult your doctor or therapist.

5.3.1. Contra-indications

The following people should consult their doctor before first use:

- People with carotid stenosis (prior to application in the area of the carotid artery/larynx)
- People with cardiac pacemakers, artificial heart valves, or cardiac arrhythmias
- People with a stent
- People with a shunt
- People with a deep brain stimulation device (DBS)
- Epileptics
- Pregnant women

The instrument must not be used:

- directly on implants (or near implants until they have completely healed)
- directly on screws
- directly on artificial joints
- directly on inflamed joints and veins
- in the case of inflammatory skin disorders
- directly on weeping eczema
- in the case of other inflammatory processes generally associated with fever
- near diseased veins
- on the stomach or back of a pregnant woman

Treatment should not commence until three days after injury at the earliest:

- Whiplash

5.3.2. Side effects

No significant medical risks are to be expected if the described precautions and contra-indications are observed.

5.3.3. Combination with other products

The NOVAFON sonic wave instrument is not intended to be combined with other products. The use of the NOVAFON sonic wave instrument on or near medical devices is not recommended.

6. TECHNICAL SPECIFICATIONS

6.1. Symbols used

~ Alternating current

○ Off (a part of the instrument only)

◉ ON (a part of the instrument only)

☐ A protection class II device in accordance with DIN EN 61140

hPa Hectopascal

Hz Hertz

KB Short-time operation

mA Milliampere

V Volts

♻ Dispose in accordance with the EU directive regarding the waste of electrical and electronic equipment (WEEE)

6.2. Technical specifications

- Room temperature: Operation from 10°C to 40 °C
- Mains supply voltage: Primary 230 V- 50 Hz 65 mA, secondary 17V- 50 Hz 450 mA
- Nominal frequency: 50 Hz
- Power supply unit: STO 20/15 min
- Nominal consumption: 9.5 W
- Manufacturer NOVAFON GmbH, Daimlerstraße 13, D-71384 Weinstadt

7. SAFETY INSTRUCTIONS

The proper and safe function of the device can be guaranteed only with the use of NOVAFON accessories. Read these safety instructions carefully prior to use:

- Handle the device carefully.
- Only store the appliance in a clean and dry place at temperatures between + 10 °C and + 40 °C.
- Check the unit before each use for signs of damage or wear.
- In each of the following cases the unit must not be used and must be repaired by a member of the customer service department:
 - > If the power cable or plug is damaged
 - > If the device does not work properly
 - > If the device is damaged
 - > If the equipment has obvious safety flaws
- Use an extension cord if the distance to the socket is greater than 3 metres.
- Before connecting the unit to the mains check whether the mains voltage corresponds with the voltage indicated on the transformer. Connect the device only to the right voltage.
- Unplug the unit from the mains by pulling out the plug, never pull on the cable.
- Never leave the unit switched on while unattended.
- Use the unit and the transformer for a maximum of 20 minutes per application, leave it to cool for at least 15 minutes afterwards.
- Do not let the power cable come into contact with hot surfaces.
- Never use the unit while bathing, showering or in potentially explosive environments.
- Never put the unit in water or in other liquids.
- Never bring the unit into direct contact with cremes, ointments, and cosmetic/

medicinal compounds.

- Avoid knocking the unit or transformer.
- Use an attachment for only one patient at any one time; resp. clean and sterilize it after each use.
- During treatment only hold the appliance by the handle or by the retractable extension.

8. MAINTENANCE AND REPAIR

8.1. Cleaning and disinfection

Disconnect the unit from the mains before cleaning. Use a cloth moistened with clean, clear water to clean the unit housing and attachments. If you use the instrument on more than one person, disinfect the accessories with a damp cloth and a non-aggressive cleaning product.

8.2. Instructions in case of defects

If defects with the unit occur, please contact our customer service team in order to discuss suitable measures.

8.3. Safety test

There is no safety check required for the NOVAFON instrument in accordance with appendix 1 of the medical devices operator ordinance (MPBetreibV). Should a routine inspection be required, we recommend a period of five years. Routine inspections and servicing are only to be carried out by authorised personnel.

8.4. Disposal

This device is subject to the EU directive regarding the disposal of waste electrical and electronic equipment (WEEE). It should not be disposed with household waste. The user is obliged to dispose of the unit in an environmentally-friendly way at a designated disposal site for waste electrical and electronic equipment. You can send your old device back to the NOVAFON company, we will happily dispose of it for you.

9. WARRANTY

From the date of delivery we guarantee—upon submission of the proof of purchase a 2-year warranty against defects that arise due to material or production faults. In case of complaint please contact your supplier directly. Initially, any right of rescission of sale (cancellation of the purchase), reduction (reduction in the purchase price) or subsequent supply is excluded. You do however reserve the statutory right to claim a reduction or cancellation should we establish a defect covered under the warranty which cannot be rectified within a reasonable period of time. In addition we provide no guarantee for damage resulting from improper use. In these cases, the liability is transferred to the operator.

Please be aware that the power cable may only be extended to a maximum length of 3

metres. Anything beyond this length requires an extension cord, as otherwise there is a risk of the cable breaking thus rendering the warranty invalid.

10. CUSTOMER SERVICE

Since the NOVAFON instrument is a finely adjusted and precise device, it may only be repaired by a member of the NOVAFON customer service department. However, in general, repair is more costly than buying a new instrument. Therefore, please do not return any faulty appliances to us unsolicited, but rather discuss the defect and the cost of repair with us first.

If, while still under guarantee, repair is no longer cost effective, the NOVAFON company will offer you a replacement (once again with a 24 month warranty).

This manual is also available as a PDF file from the manufacturer upon request.

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