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EN USER MANUAL

ORIGINAL NOVAFON INTRAORAL ATTACHMENTS

- PREFACE**
- PRODUCT DESCRIPTION**
- INTENDED USE AND APPLICATION**
- IMPORTANT INSTRUCTIONS**
- CONTRAINDICATIONS AND SIDE EFFECTS**
- CLEANING AND DISINFECTION**
- CONTACT**

1. PREFACE

The intraoral attachment heads can be used to treat speech disorders and problems swallowing. They can be used to provide both intraoral and extraoral stimulation, e.g. for tone regulation, desensitisation or to improve motor skills.

NOTE: Consult the instructions for use of the device when using the intraoral attachments. Please read the following pages carefully to ensure proper use.

2. PRODUCT DESCRIPTION

The attachments are part of DAS NOVAFON - Logo Set and are also available as spare parts. 2.1. SCOPE OF DELIVERY AND PRODUCT DESCRIPTION One of the following attachment heads is included in delivery. A description is included on the item packaging. All attachment heads are delivered with an adapter and user instructions. 2.1.1. Tongue depressor head (item no. 7003) The tongue depressor head has a spoon-like form and is used for surface stimulation. The special grooves and ripples ensure that the tongue cannot slip during application. 2.1.2. Arrow head (item no. 7004) The arrow head has three different surface structures (rippling, nubs, smooth) and its shape is specially suited for intra- and extraoral applications. 2.1.3. Spoon head (item no. 7005) The spoon head has a spoon-like trough with nubs on the bottom side. It can be used both for vibrotactile stimulation and for the application of small amounts of food. 2.1.4. Pellet head (item no. 7004) The pointed pellet head with a small ball at the end of the attachment can be used for very targeted and precise intra- and extraoral stimulation.

2.1.5. Olive roller (item no. 7024)

The olive roller has a rotatable olive-shaped oval equipped with nubs which is thus suitable for a variety of intraoral and extraoral stimulation options. The head can be removed for cleaning. 2.1.6. Pellet head XL (item no. 7026) The larger ball at the end of the attachment head extends the options for very targeted and precise intraoral and extraoral stimulation. Localised treatments inside the mouth are thus facilitated. 2.1.7. Training spoon (item no. 7027) The hole in the middle of the training spoon enables various types of tongue training. The training spoon can be used to support suction and guidance of the tongue.

2.1.8. Sensory spoon (item no. 7025)

Bolus administration can be supported through additional vibrotactile stimulation provided by the sensory spoon. The surface is also rubbed. 2.1.9. Adapter (item no. 7010) Each attachment comes with an adapter. The adapter fits on all intraoral attachments. Adapters are available as spare parts.

2.2. MATERIAL INFORMATION

The intraoral attachments are made of thermoplastic elastomers, a high-quality plastic specifically designed for medical applications. The material has no added of phthalates or latex. Please store it in a cool and dry place.

2.3. PRODUCT LIFE

The product life of the intraoral attachment can vary depending on the frequency and type of use. We recommend replacing the attachments after one year of use. Check the attachments before each application. Replace damaged, deformed or heavily discoloured products. Please note that the attachment heads are wearing parts.

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3. INTENDED USE AND APPLICATION

NOVAFON devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of

2.2. MATERIAL INFORMATION The intraoral attachments are made of thermoplastic elastomers, a high-quality plastic specifically designed for medical applications. The material has no added of phthalates or latex. Please store it in a cool and dry place.

2.3. PRODUCT LIFE The product life of the intraoral attachment can vary depending on the frequency and type of use. We recommend replacing the attachments after one year of use. Check the attachments before each application. Replace damaged, deformed or heavily discoloured products. Please note that the attachment heads are wearing parts.

3.1. INTENDED USE AND APPLICATION NOVAFON devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of

2.2. MATERIAL INFORMATION The intraoral attachments are made of thermoplastic elastomers, a high-quality plastic specifically designed for medical applications. The material has no added of phthalates or latex. Please store it in a cool and dry place.

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3.2. APPLICATION

First connect the intraoral attachment to the adapter. Then connect the adapter onto your NOVAFON sound wave device. If you wish to change or remove the attachment, simply disconnect the adapter with the attachment from the device. To remove the heads more easily, add a twisting motion while pulling. Then remove the attachment from the adapter by carefully pulling it off.

NOTE: Vibration therapy primarily relieves symptoms, but does not cure the underlying disease or replace a visit to the doctor. The use of the intraoral attachments does not replace a visit to the doctor. **NOTE:** The use of local vibration therapy for muscle relaxation in neurological diseases or voice and swallowing therapy should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

In all cases Local Vibration Therapy is used as one element in a wholistic therapy plan where muscle relaxation is needed for further exercises and training elements. **MIND THE CONTRAINDICATIONS** as named in chapter 5.

3.2. APPLICATION First connect the intraoral attachment to the adapter. Then connect the adapter onto your NOVAFON sound wave device. If you wish to change or remove the attachment, simply disconnect the adapter with the attachment from the device. To remove the heads more easily, add a twisting motion while pulling. Then remove the attachment from the adapter by carefully pulling it off.

NOTE: Vibration therapy primarily relieves symptoms, but does not cure the underlying disease or replace a visit to the doctor. The use of the intraoral attachments does not replace a visit to the doctor. **NOTE:** The use of local vibration therapy for muscle relaxation in neurological diseases or voice and swallowing therapy should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

In all cases Local Vibration Therapy is used as one element in a wholistic therapy plan where muscle relaxation is needed for further exercises and training elements. **MIND THE CONTRAINDICATIONS** as named in chapter 5.

Tongue depressor head	Tongue stimulation for muscle strengthening	Lip toning	Tongue and cheek stimulation
Arrow head	Stimulation of musculature for regulation of muscular tone	Gentle movements intra-orally or extraorally to promote perception	Tongue furrow training
Spoon head	Training of tongue tip to promote motility	Stimulation of the back of the tongue	Application of small amounts of nutrition
Pellet head	Stimulation of the anterior palatal arches	Targeting during articulation training	Median tongue furrow training
Olive roller	Gentle movements intra-orally or extraorally to promote perception	Training of tongue tip to promote motility	Tongue muscle stimulation for strengthening
Pellet head XL	Targeting during articulation training	Precise intraoral application	Stimulation of masticatory muscles and the underside of the tongue
Training spoon	Suction training	Guidance of tongue movement	Tongue tip training to promote tip lifting
Sensory spoon	Bolus administration during swallowing training	Stimulation of the back of the tongue	Treatment of swallowing issues

CAUTION! Avoid touching the teeth with the attachments, as this can cause discomfort for existing tooth infections.

4. IMPORTANT INSTRUCTIONS

4.1. HANDLING OF INTRAORAL ATTACHMENTS AND NOVAFON SOUND WAVE DEVICES

- Before using the attachments, please also observe all information from the operating instructions of your NOVAFON Intraoral sound wave device.
- Always connect the intraoral attachment to the adapter first and then attach it onto your NOVAFON sound wave device.
- If you wish to change or remove the attachment, always disconnect the adapter with the attachment from the device before you detach the attachment from the adapter. Otherwise, the device may be damaged. This applies in particular to adapters with screw threads.
- The attachments can be rotated within the adapter. If you want to rotate your attachment inside the adapter while it is already on your NOVAFON sound wave device, hold the adapter firmly with one hand while rotating the attachment. Otherwise, the device may be damaged. This applies in particular to adapters with screw threads.
- ATTENTION!** When you apply pressure with the attachment head, always hold your finger against it so that the head is safely guided and does not slip out of the adapter.
- ATTENTION!** Do not drop the attachment heads as they could be damaged. Intraoral application with one hand while rotating the attachment. Otherwise, the device may be damaged.
- ATTENTION!** Improper use may cause harm to the patient, especially during targeted application.
- ATTENTION!** If there is a biting reaction during treatment, do not pull on the attachments! Wait until the patient relaxes or force the mouth to open, e.g. using the jaw control grip.
- NOTE:** NOVAFON accessories are only intended for use with NOVAFON Intraoral sound wave devices. Use with products from other manufacturers is not permitted.

4.2. INTENDED USERS The intraoral attachments are intended for professional use only. The heads must only be used by trained medical professionals or under the strict, constant supervision of a trained medical professional.

4.3. HYGIENE Before using the intraoral attachments for the first time, they must undergo a specific hygienic treatment (according to chapter 6.) For use on several people, the attachments must be prepared hygienically after each application (according to chapter 6.). Please wear disposable gloves with each use.

5. CONTRAINDICATIONS AND SIDE EFFECTS

Observe the information on contraindications and side effects of your NOVAFON sound wave device. Due to the increased risk of infection or harm to the patient, use of the intraoral attachments is not permitted for:

- Inflammation and injury to the oral mucosa (aphthae [canker sores], thrush, bleeding, mucositis, etc.)
- Tumours in the oral cavity or head area, cancerous and not cancerous
- Bite reactions / excessive reflexes

Intraoral stimulation may occasionally result in the following side effects:

- Triggering the bite reflex (May cause harm to the user or the patient)
- Increased muscle tone, possibly to the point of causing muscle contractions and sore muscles
- Increased salivation
- Defensive behaviour

6. CLEANING AND DISINFECTION

Clean the attachment heads thoroughly.

PROCEDURE

- Cleaning and disinfection: manual
- Cleaning and disinfection: by machine

Warning information The heads must be cleaned and disinfected before the first use and after each subsequent use. There is an elevated risk of infection. Please follow the instructions below.

Restriction regarding reprocessing No special requirements.

APPLICATION location	No special requirements.
Storage and transport	It is highly recommended to transport contaminated intraoral attachments in a closed container. It is recommended to carry out the reprocessing of intraoral attachments as soon as possible, within a maximum of 2 hours after use.
Preparation	Wear personal protective equipment (gloves, water repellent lab coat, face mask, or safety glasses). Disassemble all detachable intraoral attachments into their individual parts.
Pretreatment	Ausstattung: Plastic brush (e.g. Interlock, #09098 and Interlock, #09050), running water (at least drinking water quality) <p>Procedure: Remove all detachable parts. Remove surface contamination by scrubbing all parts of the heads and the adapter (thread and cavity) under running water.</p>

MANUAL Manual cleaning	Ausstattung: Cleaning agent (multistage enzymatic cleaner; e.g. ECOLAB Sekusept® MultiEnzyme P fceolab.com), plastic brush (e.g. Interlock, #09098 and Interlock, #09050), running water (at least drinking water quality), tray for cleaning agent. <p>1. Apply parts of the attachments according to the manufacturer's instructions (e.g. 2% Sekusept® MultiEnzyme P).</p> <p>2. Immerse all parts of the attachments completely.</p> <p>3. Remove surface contamination with the plastic brush, by scrubbing all surfaces of the attachment and the adapter.</p> <p>4. Treatments with the cleaning agent according to the manufacturer's instructions of the cleaning agent (e.g. 2% Sekusept® MultiEnzyme P at least 15 min).</p> <p>5. Remove all parts from the cleaning solution.</p> <p>6. Rinse each part of the attachments under running water for 30 sec. Check for cleanliness and, if there is still visible contamination, reprocess.</p>
Manual disinfection	Equipment: Virucidal instrument disinfectant (VAH or IHO listed) based on peroxide compounds or quarternary compounds and alkalylamine/alkylamine derivatives (e.g. ECOLAB Sekusept® aktiv or Dr. Schumacher, Perfektan active), disinfectant tray <p>1. Apply the disinfectant solution according to the disinfectant manufacturer's instructions (e.g. Sekusept® aktiv, 2% / 2% Perfektan active).</p> <p>2. Submerge all parts of the attachments completely.</p> <p>3. Treatment time in the disinfectant solution according to the manufacturer's instructions for the disinfectant (e.g. at least 5 minutes for Sekusept® aktiv and at least 30 minutes for Perfektan active).</p> <p>4. Remove all parts of the attachment from the disinfectant solution.</p> <p>5. Rinse each item for 1 minute under running water.</p> <p>Wipe all parts with a lint-free disposable cloth for drying and, if necessary, dry it with medical compressed air</p>

By MACHINE

Warning information The heads must be cleaned and disinfected before the first use and after each subsequent use. There is an elevated risk of infection. Please follow the instructions below.

Restriction regarding reprocessing No special requirements.

Cleaning and disinfection: by machine	Note: Coarse surface contamination on the attachments must be removed before preparation by machine (see pretreatment): manual. Cleaning and disinfecting device (RDG) according to DIN EN ISO 15883-1+2 with thermal program (temperature 90 °C to 95 °C). <p>Cleaner: mild alkaline cleaner (e.g. ECOLAB Sekumatic® MultiClean)</p> <p>1. Place all the individual parts of the attachments in a suitable strainer bowl or in the load carrier in such a way that all inner and outer surfaces are cleaned and disinfected. Lay the parts down to avoid deformation.</p> <p>2. Close the RDG and start the programme; for programme sequence see table below.</p> <p>3. At the end of the programme, remove the products.</p> <p>4. Check the cleanliness and dryness of the load (all parts) and, if necessary,</p>
--	---

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