N O V A F O N

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ENGLISH

PHYSIO SET - INSTRUCTIONS FOR USE

- | | |
|---------------------------|------------------------------|
| 1. PREFACE | 6. PRODUCT LIFE |
| 2. SCOPE OF DELIVERY | 7. CLEANING AND DISINFECTION |
| 3. APPLICATIONS & USE | 8. CONTRAINDICATIONS |
| 4. IMPORTANT INSTRUCTIONS | 9. CE CONFORMITY |
| 5. MATERIAL INFORMATION | 10. CONTACT |

1. PREFACE

The attachments of the NOVAFON 'Physio set' (fascia ball, fascia fork fascia wedge are designed as an extension to the existing attachments supplied with the NOVAFON sound wave devices for local vibration therapy. They are intended solely for use in conjunction with such a device. Please read the following pages carefully to ensure proper use.

2. SCOPE OF DELIVERY

Delivery includes a fascia ball, a fascia fork, a fascia wedge and the instructions for use. The Physio attachments are available in anthracite gray, white or black. (item no. 7040/7044/7048)

3. APPLICATIONS & USE

The attachments are fixed to the device via a plug connection. The attachments can easily be detached by turning them a quarter turn and pulling them gently off the device. The attachments click simply onto the device. The attachments have in the correct position when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the final position. Before starting treatment, make sure that the respective attachment is engaged in the intended final position. Note: The attachments can be used directly on the skin or through fabric/clothing. There is no need for gel or similar.

3.1 INTENDED USE

NOVAFON devices and accessories are designed to relieve pains and aid in relaxation of muscles to alleviate symptoms of:

- Temporary and transient pain of the musculoskeletal system
- Chronical pain of the musculoskeletal system
- Neuro Rehabilitation (sensorimotor disorders)
- Voice and swallowing disorders⁴

3.2 TYPICAL INDICATIONS FOR USE

Fascia ball	Fascia fork	Fascia wedge
For treatment of large-scale muscle groups e.g. arms, legs and back, especially concave surfaces (crook of the arm, waist or hollow of the knee)	For targeted treatment of muscle strands and surrounding fascial tissue, greater penetration depth into soft tissue	For targeted treatment of individual muscle strands and attachments
Fascia ball	Fascia fork	Fascia fork
For targeted treatments e.g. in the neck and shoulder area	Particularly suitable for the area beside the spine [ATTENTION: Do not place attachment directly on the vertebra! Risk of injury!]	For extensive treatment of larger parts of the body and muscle strands
For relief of tension in the jaw muscles	For treatment of insensitive, extensive scar tissue	For treatment of deep-set intermuscular spaces and fascial tissue

NOTE: Vibration therapy primarily relieves symptoms, but does not cure the underlying disease. The use of the Physio-Set does not replace a visit

to the doctor.

NOTE: The use of local vibration therapy for muscle relaxation in neurological diseases or voice and swallowing therapy should only be carried out by specialists (therapists, physicians, nurses) or after detailed instruction by such a person.

In all cases Local Vibration Therapy is used as one element in a wholistic therapy plan where muscle relaxation is needed for further exercises and training elements. NOVAFON devices can be used on people from the age of 6. Children need to be supervised. The same may apply to people with limited cognition or motor control (as often found in patients with spasticity). **MIND THE CONTRAINDICATIONS** as named in chapter 8 as well as the instructions in chapter 6.

DO NOT USE for treating pains or swellings of unknown origin. In that case consult a medical professional for clarification and diagnosis.

4. IMPORTANT INSTRUCTIONS

- Depending on personal sensitivity, the vibration can be perceived with different intensity. Treat particularly sensitive spots at the lowest intensity level, through clothing or a fine cloth.
- Treat a maximum of 20 minutes per unit, then let the NOVAFON sound wave device cool down for 15 minutes.
- In case of discomfort, dizziness, or an increase in the symptoms, stop the treatment.
- Keep attachments out of reach of children.
- Do not use the attachments on mucous membranes and open wounds.
- Do not use for treating pains or swellings of unknown origin. In that case consult a medical professional for clarification and diagnosis.
- Avoid impacts! These can damage the products and hence lead to a risk of injury.
- NOVAFON accessories are only intended for use with NOVAFON vibration sound wave devices. Use with products from other manufacturers is not permitted.
- **NOTE:** When using these attachments, the noise generated may differ from that of the standard attachments. Do not switch on the device until you have made physical contact.
- Before using the attachments, please also observe all information from the operating instructions of your NOVAFON sound wave device.

5. MATERIAL INFORMATION

The NOVAFON Physio set attachments are made of high-quality plastic specifically designed for medical applications. Please store in a dry place away from sunlight.

6. PRODUCT LIFE

The product life of the attachments will vary depending on the frequency of use. Check the attachments before each application. Replace damaged or deformed products. We recommend replacing the attachments after two years of use. Even much sooner if used frequently. Please note that attachments are considered wear parts and are therefore subject to a limited warranty.

7. CLEANING AND DISINFECTION

For professional and clinical use, NOVAFON attachments must be disinfected with a surface disinfectant for medical devices before and after each application. As a general rule, NOVAFON recommends alcohol-free disinfectants that are suitable for use on sensitive surfaces, such as NOVACLEANER by NOVAFON, SinAlc sensitive by HIMED or RHEOSEPT-WD plus wipes by Rheosol. If necessary, wipe it dry with a soft cloth. In case of dirt or if otherwise necessary, clean the attachments with a brush and a neutral cleaner under running water before disinfecting. For domestic,

non-clinical use, clean the attachments with a damp cloth, under running water or in a mild soapy solution.

8. CONTRAINDICATIONS / SIDE EFFECTS

Treatments with the NOVAFON sound wave devices are not permitted for patients with:

- Open wounds/eczema [in the body region to be treated]
- Arteriosclerosis [in the body region being treated]
- Current episodes of epilepsy
- Pregnancy
- Implants [in the body region being treated]
- Heart pacemakers
- Thromboses [in the body region being treated]
- Brain pacemakers
- Cardiac arrhythmia
- Tumours, cancerous and not cancerous
- Acute inflammations [in the body region being treated]
- Acute episodes of inflammatory diseases

Side effects:

Please note that in rare cases the following side effects may occur:

- Increased pain/unpleasant sensation
- Nausea/dizziness
- Skin irritation
- Reflex/involuntary muscle contractions

Please discontinue the treatment immediately, if you experience any of these side effects. If symptoms persist, please consult a doctor. Please note that pain treatment in particular can lead to an initial worsening of the symptoms before you notice any improvement. Please note that slight reddening and warming is not a cause for concern. This is a desired effect of local vibration therapy, which is caused by the increased blood flow.

9. CE CONFORMITY

The manufacturer declares conformity with the requirements in respect to medical devices of class I according to Regulation 2017/745/EU and UK MDR 2002 in sole responsibility for this product. Please contact us in case you suspect the device to have caused a serious incident. As a user you have the right to report such an incident to your national competent authority [e.g Ministry of health; FDA].

10. CONTACT

If you have any questions, please contact your local dealer or the manufacturer.

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NOVAFON Elektromedizinische Geräte GmbH,